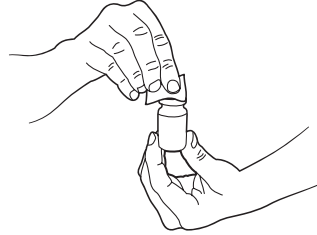
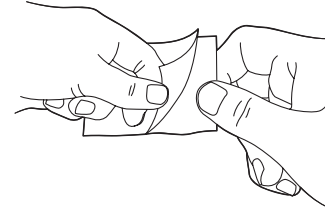
**1.**

Wash your hands well with soap and water, then dry them completely. Put on gloves if necessary.

**2.**

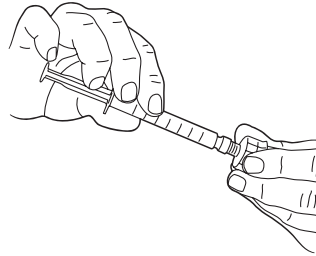
Remove the plastic cap from the medication vial and use an alcohol wipe to clean the rubber stopper.

**3.**

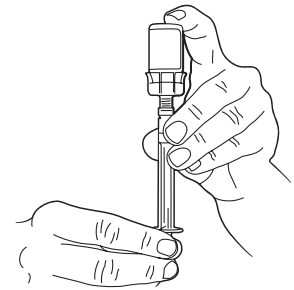
Remove the back part of the vial adapter package, making sure that the adapter stays in the front part.

**4.**

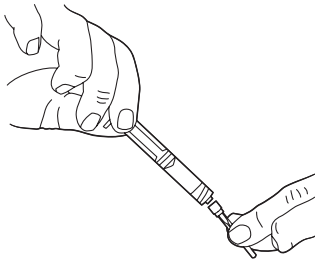
Using the front part of the package to handle the adapter, connect the adapter to the vial by pushing it down onto the top of the vial until the adapter snaps in place.

**5.**

Disinfect the vial adapter top with an alcohol wipe before each use. Using only the syringe without needle attached, pull back the plunger to the prescribed amount for one injection then twist the syringe onto the top of the vial adapter.

**6.**

Flip the vial, adapter and syringe upside down and push the plunger in. Draw back the liquid medication into the syringe to the prescribed amount for one injection. Untwist the syringe from the vial adapter.

**7.**

Twist a capped needle onto the syringe. Follow the injection technique described in the accompanying instructions. Store the vial and attached adapter as indicated on the vial label or in the accompanying medication information. Repeat steps 5 and 6 before each injection.